IMPORTANT NOTICE - INSTRUCTIONS FOR USE

EN

CONTAINER LABEL DEFINITIONS

ECP	Eye Care Professional
BCR	Base Curve
TD	Diameter
BVP	Power
Cyl	Cylinder Power
Axis	Axis of cylinder
ADD	Additional Power
	Device has been sterilised using steam
	Batch code
	Caution
	Lens expiry date (year and month)
	Manufacturer
	Do not use if tamper-evident seal is damaged
	For single use only
	Product meets the standards set in the European Medical Device Regulation
	Authorised representative in the European Community
	Importer
	Medical Device
	Consult instructions for use
12/12	12-month modality

This leaflet contains important **PRODUCT & SAFETY** information – please read carefully and retain for future reference.

Contact lenses should always be fitted by an Eye Care Professional (ECP) on non-diseased eyes.

It is essential that you follow your ECP's directions and all labelling instructions for proper use of your contact lenses.

The contents of contact lens packaging remain in a sterile condition until opened.

Should the tamper evident seal be broken, **DO NOT USE** the lens and return it to your ECP at the earliest opportunity.

PRODUCT DESCRIPTION

Soft contact lenses supplied sterile for single patient use and intended for daily wear. Packaged in vials or blisters containing 0.9% borate buffered saline (b blister packaging also contains Poloxamer 407NF).

INDICATIONS FOR USE

Vision Correction

Contaflex 77 soft contact lenses Indicated for the optical correction of refractive ametropia (**myopia** and **hyperopia**) in **phakic** or **aphakic** persons with up to approximately 0.75D (dioptries) astigmatism that does not interfere with the visual acuity.

MODALITY & DISCARD DATE

Hydrogel

Soft contact lenses manufactured from hydrogel material listed below have a recommended modality of up to 12months wear. They should be discarded 12months after opening or on the expiry date, whichever come first.

Hydrogel materials:

CONTAFLX clear (filcon 2, [77%])

CONTAFLX tint (filcon 2, [77%])

LENS DISPOSAL

Contact lenses can be recycled through a dedicated recycling programme. Otherwise, they should be disposed of in general waste. Do not flush down the toilet or dispose of down the sink.

CONTRAINDICATIONS

DO NOT USE the contact lens when any of the following conditions exist:

- Acute and sub-acute inflammation or infection of the anterior chamber of the eye.
- Any eye disease, injury, or abnormality that affects the cornea, conjunctiva, or eyelids.
- Severe insufficiency of lacrimal secretion (dry eyes).
- Corneal hypoesthesia (reduced corneal sensitivity).
- Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses.
- Allergic reactions of ocular surfaces or adnexa (surrounding tissue) that may be induced or exaggerated by wearing contact lenses or use of contact lens solutions.
- Allergy to any ingredient, such as mercury or Thimerosal, in a solution which is to be used to care for the contact lens.
- Any active corneal infection (bacterial, fungal, or viral).
- If eyes become **red** or **irritated**

PRECAUTIONS

Follow the guidance below to reduce the risk of microbial infections

Do not allow non-sterile water (e.g. tap-water) to come into contact with the lens or lens cases due to increased risk of microbial infection.

- Swimming or water sports should be performed only when water-tight goggles are worn to prevent contamination due to increased risk of microbial infection.
- Do not shower or bathe whilst wearing soft contact lenses.

Follow the guidance below for optimal contact lens performance

- Use only those solutions and products recommended by your ECP when cleaning your lenses and cases.
- Do not change the contact lens care system without consulting your ECP.
- If your lenses have not been worn for a long period of time, follow the solution instructions before use.
- Do not exceed the wearing time schedule advised by your ECP.
- Always maintain the recommended replacement schedule and return for the aftercare visits advised by your ECP.
- Certain medications may affect your ability to wear contact lenses. Advise your ECP of any new or changes to existing medications.
- Do not change to a different lens type or change lens parameters without consulting an ECP.

- In the event of redness of the eye, excessive lacrimation, pain or blurring of vision that is not attributable to contaminated lens surfaces, remove your lenses from the eyes and consult your ECP as soon as possible.
- Do not expose the contact lenses to hairsprays, cosmetics and aerosol sprays.
- Do not sleep in your contact lenses unless your ECP recommends it.
- Multifocal contact lens wear may lead to nighttime glare and haloes or reduced contrast sensitivity. If you experience these symptoms, consult your ECP.

- If your soft contact lenses have not been worn for a long period of time, follow the solution instructions before use.
- If lens becomes completely dehydrated, do not attempt to rehydrate and discard the lens.
- Store at room temperature
- Ensure you distinguish between the right and left lens.

WARNING

Remove contact lenses immediately if they come in contact with noxious vapour, e.g. chemical of hazardous substances or you are in an environment with hazardous fumes.

In the event of redness of the eye, excessive lacrimation, pain or blurring of vision that is not attributable to contaminated lens surfaces, remove your lenses from the eyes and consult your ECP as soon as possible.

UV filters contained in contact lens materials do not fully protect your eyes from UV radiation and cannot protect the whole eye. It is recommended to use sunglasses that are verified as blocking UVA and UVB to give the best protection from the sun.

INSERTION AND REMOVAL INSTRUCTIONS FOR PATIENTS

ATTENTION TO PERSONAL HYGIENE IS VERY IMPORTANT. ENSURE THAT FINGERNAILS ARE KEPT AS SHORT AS POSSIBLE.

Wash hands with warm water and soap, drying hands thoroughly before removing the lens from its container. Use a clean and bright environment to handle your contact lenses.

- Remove the lens from its container, ensure that it is not inside out and place the lens on the tip of the forefinger. Look into a mirror and, with the middle finger of the same hand, draw down the lower lid.

- With the middle finger of the free hand, pull the upper lid upwards and place the lens gently over the visible iris. Release the lids slowly.

LENS REMOVAL

Wash hands with warm water and soap, drying hands thoroughly before removing the lens from its container.

Applying downward pressure on the lower lid whilst looking up will help move the lens downwards. Then gently ease the lens onto the 'white' of the eye using the fingertip. The lens can then be pinched off the eye using the thumb and forefinger.

Following removal, clean and store your contact lenses in accordance with the instructions provided by the manufacturer of the solution and your ECP.

This should be normal shortly after insertion but, if poor vision persists check that the lenses are not inside out or in the wrong eyes.

COMFORT

If the lens is uncomfortable, it may be necessary to remove the lens for cleaning, rinsing and re-insertion. Again, if the lens is inside out, discomfort may be felt and the lens will require removal, reversal and cleaning prior to re-insertion.

Should discomfort persist, remove the lens and do not wear it again before consulting your ECP. Ensure your ECP notifies the manufacturer of the contact lens or authorised representative in the case of a serious

Do not exceed the wearing time schedule advised by your ECP.

Information specific to your contact lens(es) can be found on the contact lens packaging. This information contains the following:

- Contact lens prescription
- Trade name of the contact lens design
- Material in which the lens is manufactured from
- Contact lens expiry date

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