YOUR EYES ARE PRECIOUS
Please read about your new contact lenses
Instruction for use
IMPORTANT

DESCRIPTION

SEED 1dayPure moisture shows spherical lens design and is available as a spherical lens for single vision.

SEED 1dayPure moisture Bifocal shows bifocal lens design and is available as an aspherical bifocal lens.

SEED 1dayPure moisture for Astigmatism shows toric lens design and is available as an astigmatic (toric) lens for single vision.

These lens materials are based on a 2-HEMA compound where positive and negative ions co-exist. The lens is immersed in isotonic buffered saline with Polaxamer 407 and alginic acid.

The water content is 58% and the oxygen permeability coefficient is 30.0×10⁻¹¹ (cm²/sec) (mLO₂/ (mL×mmHg)). The base curve is 8.8 and the index of refraction is 1.406. Borates (boric acid & sodium borate) in isotonic buffer saline are classified as CMR 1B substances in a concentration above 0.1% weight by weight and are safe when product is used according to label instructions.

INTENDED USE

Correction of vision

INDICATIONS

Spherical lens is indicated for myopia, hyperopia.

Aspherical bifocal lens is indicated for myopia, hyperopia, and presbyopia. Astigmatism (toric) is indicated for myopia, hyperopia and astigmatism.

INSTRUCTIONS OF USE

These lenses are daily wear, daily disposable soft contact lenses. Please wear lenses only while awake and remove them before sleeping. Once daily disposable lenses have been removed from the eye, it must be thrown away.

• **Instructions on handling lenses**

1. Keep your fingernails short and smooth to avoid scratching your eye or your lenses.
2. Always wash your hands with soap and rinse carefully before handling your lenses. Lenses may become contaminated if you do not wash your hands properly.
3. Always handle your lenses somewhere that is well-lit, clean and sterile.
4. Your contact lenses may become damaged depending on how you handle them.

• **Checking the lenses and blister packaging**

1. Check which lenses are for the right eye and which lenses are for the left eye.

2. Check if there is anything wrong with the lenses and blister packaging before opening. If there is anything wrong, for example, if there is any damage or leakage, do not open and contact the point of purchase.

• **Removing the lenses from the blister packaging**

1. Break off the blister packaging by folding down the sides.
2. Remove the label, keeping a firm hold on the blister packaging and the label. When doing this, be careful not to cut your hand on the blister packaging or label. Also note that the lens storage solution inside the blister packaging may splash upon removal of the label.
3. Remove the lens from the blister packaging by sticking the lens to the pad of your index finger. Alternatively, turn the blister packaging upside down and empty the lens together with the storage solution into the palm of your hand. Take care, because if you dig your nail into the lens or scrape the lens on the rim of the blister packaging on removal, the lens may be damaged.

• **Checking the lenses**

1. Hold the edges of the lens from the side and gently move up and down to remove the liquid.
2. Put the concave surface on the index finger of your dominant hand and check which the right and wrong sides of the lens are. A lens has a right side and a wrong side.

Ensure that the lens is not inside out. Place the lens on your right index finger and verify the shape. (fig. 1 and 2)

fig. 1

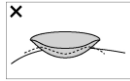
fig. 2

3. Check if there is anything wrong with the lens, for example, deformation, discolouration, foreign matter, damage, scratches and dirt, and do not wear the lens if there is anything wrong with it.

• **Inserting lenses**

1. Put the lens onto your finger, and, with the middle finger of the same hand, pull down your lower eyelid. With the index finger of your other hand, pull up your top eyelid. If you put your finger on your eyelash line, you can open your eyes wide.
2. Look in the mirror so that your cornea comes into the center of your eye. Slowly bring the lens close to your eye and put it gently in place, still looking in the mirror. If you glance upwards when doing this, the lens will not be properly centered on your cornea.
3. When you have placed the lens properly on the cornea, slowly remove your index finger and then remove your fingers which had been holding your eyelids, starting with the bottom lid first and then the top lid. If you remove your fingers quickly and blink your eye shut, the lens may change position or fall out.
4. Check in the mirror that the lens is on top of your cornea and, covering the opposite eye with your hand, check your sight in your right and left eyes.

5. Follow the same procedure to fit the lens in your other eye.

• **Removing the lenses**

1. Check the position of the lens in the mirror.
2. Looking straight in the mirror, hold the mirror upward and look at the eye reflected in the mirror.
3. With your middle finger, slowly pull your lower lid down, and with the index finger of the same hand, move the lens down from the cornea.
4. In this position, use the pads of your thumb and index finger to gently take hold of the bottom of the lens.

To avoid scratching your eye, take care not to touch your eye directly with the tips of your fingers or your fingernails.

If you are unable to remove the lens, blink several times and then try again.

CONTRAINDICATIONS

1. Medical conditions

Acute and sub-acute inflammation of the anterior eye, Eye infection, Uveitis, Reduced corneal sensitivity, Dry eye or lacrimal apparatus disease to an extent that poses a problem for wearing lenses, Eyelid disorder, Allergy to an extent that affects the wearing of lenses, Other diseases which would hinder use of contact lenses.

2. Environmental conditions

Anyone living or working in dry environments.

Anyone living or working in an environment where dust or chemicals are likely to enter the eye.

3. Personal qualities

Anyone who cannot follow the recommendations of their eye care practitioner, anyone who cannot use contact lenses properly, anyone who fails to attend regular check-ups, anyone who cannot follow the hygiene routine required to wear contact lenses, anyone who has an extremely nervous disposition.

4. Prohibition of reuse

A lens cannot be reused once it has been removed from the eye.

WARNINGS

- Even when used properly, wearing contact lenses may cause eye disorders such as the following, and if left untreated, these eye disorders may become serious or may result in loss of sight. To wear your contact lenses safely, be sure to read this Instructions for Use carefully and to follow the instructions of your eye care practitioner and the proper methods for handling contact lenses.

Example for eye disorders

- Corneal edema • Corneal ulcer • Corneal infiltration • Corneal epithelial erosion
- Keratitis (including infectious keratitis) • Conjunctivitis (including giant papillary conjunctivitis)
- Possibility of speeding up reduction in corneal endothelial cells
- Possibility of changing the shape of the cornea
- Lenses may be damaged or become damaged. If you have inadvertently worn a lens which is damaged or otherwise faulty, or if a lens becomes damaged while you are wearing it, remove the lens immediately and consult your eye care practitioner immediately, regardless of whether you have any subjective symptoms.

PRECAUTIONS

Be sure to read this Instructions for Use carefully before using your contact lenses, and if you are in any doubt about the terms or information contained herein, please consult your eye care practitioner for clarification before using the lenses.

Retain this Instructions for Use for future reference.

1. Avoid direct sunlight and avoid freezing, store at room temperature.

SEED Co., Ltd.

2. Do not wear contact lenses that are damaged.
3. Do not use contact lenses past their expiration date. This lens must be replaced daily, and do not reuse once it has been removed from your eye.
4. Contact lenses can be disposed of as regular household trash (Do not flush contact lenses down the sink or toilet).
5. Do not wear contact lenses for longer than the recommended wearing period.
6. Use contact lenses properly in accordance with recommended methods for handling contact lenses.
7. Always attend regular contact lens check-ups.
8. If you experience any problem at all, consult your eye care practitioner immediately.
9. Always insert lenses promptly after opening, as lenses may become contaminated.
10. Avoid rubbing your eyes while wearing lenses, as this may result in scratches to your eyes or displacement of the lenses.
11. Do not mix up right and left lenses, as doing so may result in impairment of vision and discomfort.
12. Make sure you have appropriate glasses to use in case you are unable to wear your contact lenses due to the condition of your eyes or for other reasons, or after you have worn your lenses for the wearing period recommended by your eye care practitioner. Continuing to wear your contact lenses regardless may cause eye disorders or other problems.
13. Do not wear contact lenses for driving or operating machinery until you are used to them.
14. If a lens becomes decentered or falls out while you are driving or operating machinery, stop driving or operating the machinery promptly after making sure that it is safe to do so.
15. Follow the instructions of your eye care practitioner if using eye drops while wearing contact lenses.
16. Do not allow cosmetics, pharmaceuticals or other substances to come into contact with your lenses. If cosmetics (eyeliner, cleansing products, etc.) become attached to your lenses through your tears, this may result in discolouration or deterioration in the quality of the lenses, making them unusable.
17. When your eyes are exposed to strong wind (when riding a motorbike or bicycle, skiing, in strong wind, etc.), always use some kind of protection against the wind such as sunglasses or goggles. Failure to do so may result in uncomfortably dry lenses or the loss of a lens.
18. Consult the eye care professional about wearing lenses during sporting and water related activities. Exposure to water while wearing contact lenses in activities such as swimming, water skiing, and hot tubs may increase the risk of ocular infection, including but not limited to Acanthamoeba keratitis.
19. Never allow contact lenses to come into contact with non-sterile liquids (including tap water and saliva) as microbial contamination can occur, which may lead to permanent eye damage.
20. Always remember to take spare lenses and glasses with you when you go on a trip, etc. in case you lose or damage your lenses.
21. Never wear someone else's lenses or give your lenses to someone else, as this may result in problems such as use of the wrong lenses or eye disorders.
22. Do not wear on top of other lenses, as doing so may result in problems such as eye disorders.
23. Users who find it difficult to insert or remove contact lenses themselves such as children or elderly people should consult their eye care practitioner for appropriate guidance.
24. Pregnant women may experience some difficulty wearing contact lenses that were once comfortable or experience a temporary deterioration in their vision, and they should seek appropriate guidance where necessary.
25. People who suffer with allergies may be more prone to diseases of the eyes and skin than other users, and they should seek appropriate guidance where necessary.
26. Use of contact lenses may be affected by illness, pregnancy, childbirth, use of medications, physical condition, eye conditions, and living environment. The conditions in which you wear contact lenses may also change after you have started wearing contact lenses. Always seek appropriate guidance where necessary.
27. Always check the expiry of the contact lenses before wearing if they have not been worn for a length of time.
28. Consult your eye care professional if you wish to change the lens type or parameters.

POTENTIAL ADVERSE EFFECTS

—Until you are used to your lenses—

Until you are used to your lenses, you may experience symptoms such as the following. Symptoms vary from person to person but will gradually diminish as you become used to your lenses. However, if you experience acute symptoms or if symptoms persist, stop wearing your lenses and promptly consult your eye care practitioner.

- Slight discomfort • Slightly bloodshot eyes • Unstable vision • Watery eyes • Itchy eyes

WEARING SCHEDULE

1. This is a daily wear, daily disposable contact lens. Please wear the lenses only while awake and remove them before sleeping.
2. The wearing and replacement schedules should be determined by the eye care professional as individual response to contact lenses varies. Regular checkups or follow ups, as determined by the eye care professional, are also very important. (Suggested Schedule)

Day	1	2	3	4-6	7 and after
Hours	6	8	10	12	All waking hours (12~14)

3. In cases where contact lens wear is stopped for:

- **Less than a week** - please start by shortening the wearing schedule by 2 to 4 hours.
- **More than a week** - please start by referring to the table above.
- **More than 1 month** - please consult your eye care professional before starting the contact lens wear.

CARE FOR A STICKING (NON-MOVING) LENS OR TORN LENS

If the lens sticks(stops moving) or cannot be removed, apply a few drops of the recommended lubricating solution directly to the eye, blink and wait until the lens begins to move before removing it. If nonmovement of the lens continues, immediately consult your Eye Care Practitioner. If the lens tears in the eye, it will cause discomfort. Carefully take out the torn lens and throw it away. If there is a small piece of the lens left in the eye, rinse the eye with sterile saline solution and blink to dislodge it. Avoid excessive pinching. Contact your Eye Care Practitioner for assistance if you can't locate a part of the lens or if you can't remove it.

EMERGENCIES

Patient should be informed that if chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes, the patient should: Flush eyes immediately with tap water or fresh saline solution, remove and discard the lens, and immediately contact the eye care professional or visit a hospital emergency room without delay.

REPORTING OF INCIDENT

Any incidents from use of SEED contact lenses should be reported to the manufacturer and/or its authorized representative and the relevant competent authority.

SEED Co., Ltd. Kounosu Laboratory
1030-7, Fukuro, Kounosu, Saitama, 369-0131, Japan

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UltraVision International Ltd
Commerce Way, Leighton Buzzard, Bedfordshire, LU7 4RW, United Kingdom

Patent <https://www.seed.co.jp/patents/>

Rev.07/2024

Meaning of symbols

Symbol	Meaning	Symbol	Meaning	Symbol	Meaning	Symbol	Meaning
P	Power	BC	Base curve	DIA	Diameter	EC REP	Representative of EC
Lo	Low depth of focus	Mid	Middle depth of focus	Hi	High depth of focus	LOT	Batch code
Manufacturer icon	Manufacturer	Do not reuse icon	Do not reuse	Use by date icon	Use by date	STERILE icon	Sterilized using steam or dry heat
Caution icon	Caution	Consult Instructions for Use icon	Consult Instructions for Use	EDOF	Extended depth of focus	CH REP	Representative of Switzerland
Importer icon	Importer	MD	Medical Device	UK REP	Representative of UK	Date of manufacture icon	Date of manufacture
Do not use if package is damaged icon	Do not use if package is damaged	Single sterile barrier system icon	Single sterile barrier system	Hazardous substances icon	Contains hazardous substances		

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Wöhik Contactlinsen GmbH