YSO 38% SOFT CONTACT LENS

CONTENTS:

sterile hydrophilic contact lens for monthly disposable in buffer saline solution either alone, or containing Sodium Hyaluronate and Trehalose, or PMB wetting agent.

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CAUTION:

Consult your eye care practitioner for lens wearing and care instruction.

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INSTRUCTIONS FOR CONTACT LENS WEARER:

- 1.The lens is to be dispensed to individual prescriptions only. Use the lens as instructed by your optician. Consult your practitioner on a regular basis for advice on recommended changes.
- 2.Wash and dry your hands accurately before inserting or removing the contact lens.
3. The monthly disposable lens is to be cleaned, rinsed and disinfected each time it is removed from the patient's eye and discarded after the recommended wearing period prescribed by the eye care practitioner. Do not sleep in contact lens.
- 4.Do not use the contact lens after the date printed on the lens-pack. Do not use lens from damaged lens-packs.
- 5.Never use contact lens when experiencing eye dryness, eye inflammation or eye disease. Please remove the contact lens and contact your optician immediately if you experience eye discomfort or any other problem after inserting your contact lens.

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